

DICOM send to Onepacs Gateway

DICOM C-STORE to a transmission computer (OnePacs Gateway)

This is the “workhorse” method of transmitting studies to OnePacs that will be used most commonly when studies are being transmitted directly from imaging modalities to the OnePacs system. In this method, a computer (gateway PC) is placed on a local imaging facility network, and the imaging modalities are configured to transmit studies directly to that PC over the local network. On receipt of the studies, the gateway compresses and encrypts the studies, then uploads them to OnePacs.

Further details can be found [here](#).

If you have many studies on physical media to upload to OnePacs, please see the section on [importing studies via the OnePacs Gateway](#).