

Viewers for the iPad

Viewers for the iPad


Cases may be viewed in the [web viewer](#) or [Osirix HD™](#).*

- The [web viewer](#) is available for the iPad. Please note that physical memory may have an impact on performance.
- [Osirix HD™](#) is a compatible viewer for the iPad. This is available in the App store. Note that as of this writing, Osirix HD is not intended or approved for diagnostic use.

To view a case on the iPad in Osirix HD™, open the OnePacs worklist in Safari, then click on the toolbar menu icon for the case you wish to view. A menu will appear with an option to open the case in Osirix HD™. This will download the entire case to Osirix HD.



In the event that the case is large and you wish to view only a single series, to avoid having to download the entire case, you may click the "series preview" icon on the toolbar, and select a specific series to view, and click on the "View in Osirix" icon which appears in the series preview pane.

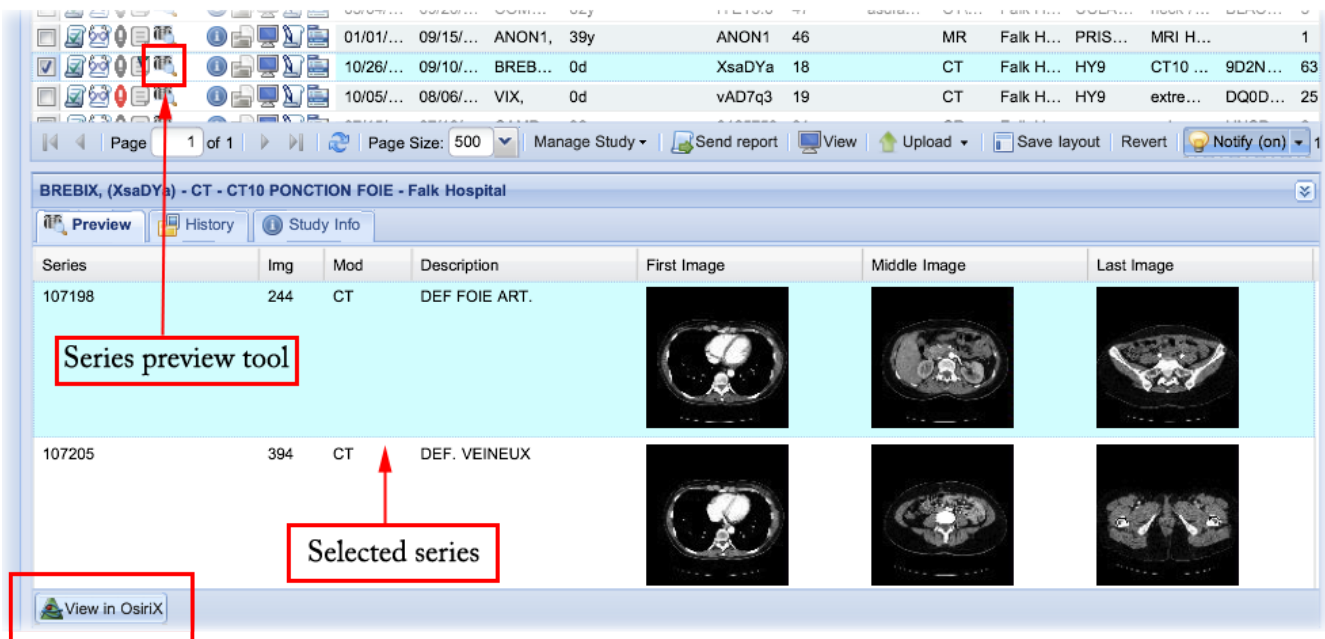

Upcoming improvements

Upcoming improvements to iPad support include:

- Integration with other app-based iOS viewers is currently under investigation.
- A variety of tweaks and modifications to improve the iPad experience.

Feedback welcome

We are interested in user experience reports to aid us in tailoring OnePacs for use on mobile devices.

If you have accessed OnePacs through an iPad, Android, or other mobile device and have feedback to offer, please feel free to offer such feedback through the [OnePacs User Forums](#)

**OsirixHD™ is a trademark of Pixmeo SARL. This company and its products are not affiliated with OnePacs. No endorsement of this companies or its products by OnePacs, or endorsement of OnePacs by this company, is implied.*